**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D95448 Bayreuth**  
**Germany**  
**T +49 921 9120**  
**F +49 921 912780**  
**ortho@medi.de**  
**www.medi.de**

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium bvba  
 Posthoornstraat 13/1  
 3582 Koersel  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Brasil  
 Rua Cristovan de Vita, n.º 260, Un. 30,  
 Centro Logístico Raposo Tavares,  
 Vila Camargo  
 Vargem Grande Paulista - SP  
 CEP 06730-000  
 Brasil  
 T: +55-11-3500 8005  
 sac@medibrasil.com  
 www.medibrasil.com

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbaek Strand  
 Denmark  
 T +45-70 25 56 10  
 F +45-70 25 56 20  
 kundeservice@sw.dk  
 www.meditandmark.dk

medi Bayreuth Espana SL  
 C/Canigo 2-6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34-932 60 04 00  
 F +34-932 60 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33-1 48 61 76 10  
 F +33-1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1 371-0090  
 F +36 1 371-0091  
 info@medi.hu  
 www.medi.hu

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31-76 57 22 555  
 F +31-76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi Orient M.E.A. ZCC  
 TAURUS GROUP  
 Suite No. M01 Wasel Al Mamzar Bldg,  
 Al Wuhida Road, Al Mamzar  
 P.O.BOX NO. 91464  
 DUBAI, UAE  
 United Arab Emirates  
 T: +971-4 2557344  
 F: +971- 4 2557399  
 md@taurusgroup.net

medi Polska Sp. z o.o.  
 Zygmunt Starego 26  
 44-100 Gliwice  
 Poland  
 T +48-32 230 60 21  
 F +48-32 202 87 56  
 info@medi-polska.pl  
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda  
 Rua do Centro Cultural, no. 43  
 1700-106 Lisbon  
 Portugal  
 T +351-21 843 71 60  
 F +351-21 847 08 33  
 medi.portugal@medibayreuth.pt  
 www.medi.pt

medi RUS LLC  
 Business Center NEO GEO  
 Butlerova Street 17  
 117342 Moscow  
 Russia  
 T +7-495 374 04 56  
 F +7-495 374 04 56  
 info@medirus.ru  
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.  
 Unit 1806, Floor 18, Sandhill Plaza  
 No. 1  
 2290 Zuchongzhi Road  
 200131 Shanghai  
 People's Republic of China  
 T: +86-1366 1668 687  
 F: +86-2161 7696 07  
 info@medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis  
 Tic. AS  
 Kültür mah. çaldıran sok.14/3  
 06420 Çankaya/Ankara  
 Turkey  
 T +90 312 435 20 26  
 F +90 312 434 22 82  
 info@medi-turk.com  
 www.medi-turk.com

medi UK Ltd.  
 Plough Lane  
 Hereford HR4 0EL  
 Great Britain  
 T +44-1432 37 35 00  
 F +44-1432 37 35 10  
 enquiries@mediuk.co.uk  
 www.mediuk.co.uk

4 046114 561996

E012145 / 08.2018

## Lumbamed® disc

Lumbalstützorthese · Lumbar orthosis · Ceinture de soutien  
 lombaire · Órtesis de soporte lumbar · поясничный бандаж

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso.  
 Instruções para aplicação. Istruzioni per l'uso.  
 Gebruiksaanwijzing. Brugsvejledning. Bruksanvisning. Návod k  
 použití. Uputa za uporabu. Инструкция по использованию.  
 Kullanma kılavuzu. Instrukcja zakładania. Οδηγία εφαρμογής.  
 Használati útmutató. Uputstvo za upotrebu. Інструкція з  
 використання. Návod na použitie. دليل الاستخدام. 使用说明.  
 Instrucțiuni de utilizare. שימוש הוראות.

medi. I feel better.

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### Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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# Lumbamed<sup>®</sup> disc

## Intended purpose

The Lumbamed disc is a lumbar support with adjustable stabilisation system. The product is exclusively to be used for the orthotic fitting of the lower back and only on intact skin or if any wounds have been properly covered.

## Indications

- Slipped disk, disk protrusion (including after operations)
- Repeated blockages in the lower part of the back
- Pain due to attrition (e.g. arthrosis of the vertebral joints)
- Muscular imbalance of the abdomen/back

## Side Effects / Contraindications

No side effect is known which influences the whole body. However, if supports (or similar aids) are too tight, this may lead to local pressure effects or the constriction of blood vessels or nerves in individual cases. You should thus consult your doctor in the following circumstances and before applying the support:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).

Any combination with other products must be discussed in advance with the responsible doctor.

## Mode of operation / note on wearing

Lumbamed disc stabilises the lower thoracic and lumbar spine area as well as the transition to the pelvis without impairing the muscular power, using a combination of compressing knitted fabric, internal back rods and an external strap supporting system. The

statics of the spine are improved and the spinal segments concerned are relieved, which reduces pain. Variable supporting elements make a quick mobilisation possible. The stability of the brace can be adjusted to the process of recovery by removing the strap supporting system (cf. Fig. 11). Moreover, the internal back rods can be replaced by more flexible rods (available as an option, cf. Fig. 12). This activates the trunk musculature step by step. Your attending physician will specify the corresponding steps of therapy.

In principle, the support can and should be worn all day at the beginning of the therapy and in particular after operations to support the success of the treatment. The special cut in the area of the groin (Comfort Zone) is decisive for the correct and comfortable fit of the support. If you suffer excessive pain or discomfort during wear, please take off the product immediately and seek medical advice. The support is suitable for use in water. After use, please rinse out the product thoroughly.

## Putting on the brace

- Before putting on the orthotic, please loosen the straps, so that it can be put on without pulling. To do so, unfasten the velcro ends of the straps and loosen these by placing them further back on the velour on the belt (Fig. 1).
- The orientation of the label on the righthand inner fastener has information on how to put on the brace (Fig. 2).
- Now, slide your left hand into the inner hand strap on the lefthand fastener side (Fig. 3).
- Wrap the brace around your body and check once again on the label that you are holding the brace the right way (Fig. 4).
- Now, slide your right hand into the outer hand strap on the righthand

fastener side (Fig. 5).

- Pull both fastener sides together in the middle, so that they can be velcroed over each other as far at the same distance. When fastening, ensure that the back brace is in the middle. The orthotic is on properly when the lower fastener edge is just above the pubic bone (see Fig. 9) (Fig. 6 + 7).
- Now unfasten the velcro ends of the pull straps and pull these forward at the same time. You can adjust the tightness to suit your own requirements and manage your pain levels (Fig. 8).
- Information for technicians: The strap length may be adjusted by cutting it at the edge of the Y fastener (Fig. 10).

Please take care that trained staff instruct you when putting on the support for the first time.

### Removing / Changing the stabilising rods

Hold the lower end of the rod and pull it slightly downwards with the knitted fabric. At the same time, slightly pull apart the upper opening in the knitted fabric. You can now remove the rod easily from the pocket. For inserting the rods, slide them into the individual pockets again and reposition them in the same way.

---

### Washing instructions

Before washing, please remove the internal rods (cf. Fig. 12) as well as the strap supporting system by detaching it from the Y connecting piece (cf. Fig. 11). During replacement make sure that the medi logo on the back stays is at the top. Velcro tabs must be closed for washing. We recommend using a laundry bag. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and

material damage. You can wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.

- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

### Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

### Composition

Polyamide, Polyester, elastan, ABS Plastic

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

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